

STERISCAN™

STEAM STERILIZATION INTEGRATOR

Takes The Guesswork Out Of Steam Sterilization



Unprocessed



Processed
Reject




Processed
Accept



If the dark bar has not reached the *accept window* adequate sterilization conditions are not satisfied. Material should be reprocessed. *Product shown actual size: 3/4" x 4"*.

SteriScan™ Integrator parallels the performance of biological indicator spore strips with a safety margin throughout the entire sterilization range.

Accurate and Instant Results with SteriScan.

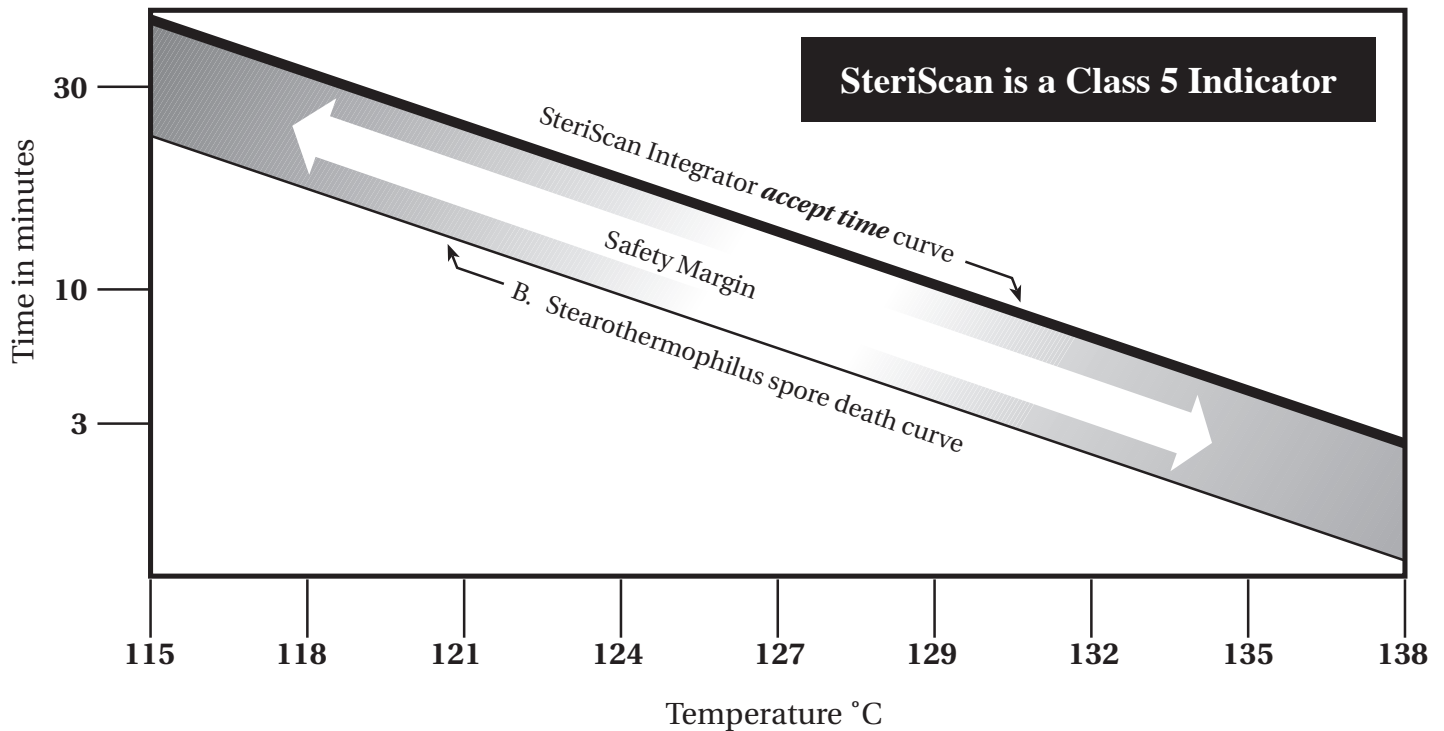

Accurately Integrates All Critical Steam Sterilization Parameters

- Time
- Temperature
- Steam

Quality You Can Depend On

- Developed by the inventor of the original integrator.
- Made with safe and non-toxic chemicals.

Easy To Read

- Advancing color bar.
- Not a color changing strip.
- Eliminates the need to interpret or match colors.

Effective Across The Normal Sterilization Range (115°C to 138°C)

- Eliminates the need to carry multiple inventories.

Distributed by:

Reorder Number 1100

Packaged 250 units in a foil bag and 8 bags (2,000 units) to a case.

Dana Products
11457 Melrose Street
Franklin Park, IL 60131
Phone: 847-455-2881
Fax: 847-455-2886