


VERIFY Bowie Dick Indicator Strips:

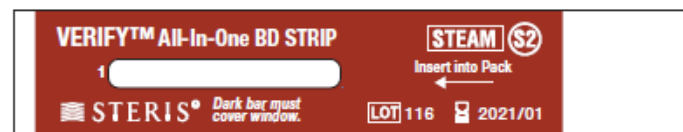


The reusable test packs use the VERIFY Bowie Dick Indicator Strips to challenge the air removal efficacy of prevacuum steam sterilizers operating at 270°F. Either the sterilizers preprogrammed Daily Air Removal Test Cycle (Bowie Dick Test Cycle) or a prevacuum cycle operating at 134°C/273°F with a 3.5 minute exposure and a 2 minute dry time may be used.


During the test cycle, the steam sensitive chemical located within the strip will wick across the viewing window. The dark color will completely fill the window if all air was removed from the reusable test pack and replaced by steam.



UNPROCESSED

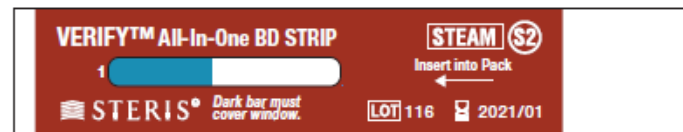

PROCESSED REJECT

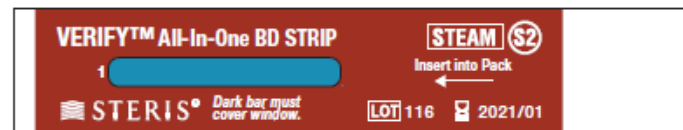

PROCESSED ACCEPT

DARK BAR MUST COMPLETELY COVER WINDOW